



Guide 3

Hands-free and accessibility


Answer and speak with a look, a switch, a nod, or a blink.

Accessibility is the point of Rejoin, not an add-on. Everything here runs on the device, and each option is off until it is turned on in Me, under Accessibility and display.

For people who cannot reliably use their hands, and their caregivers.

1. Big Yes and No answers


When a yes-or-no question comes up, Rejoin can show two large answer anchors. They can be triggered by eye tracking, an external switch, a head nod or shake, a deliberate blink, even a hum, whatever movement is still reliable.

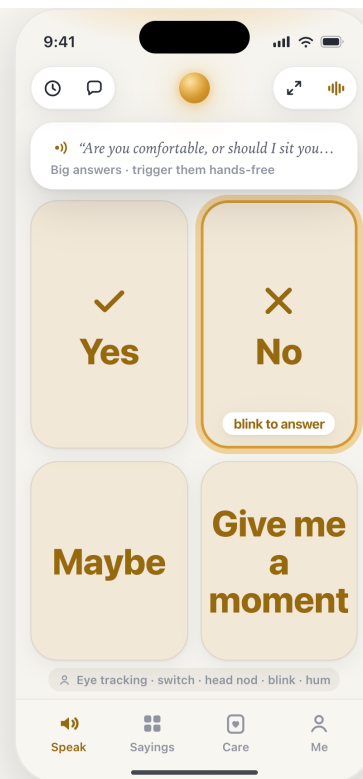

Yes / No anchors.

2. Dwell to select

With eye tracking or a hovering pointer, rest on a tile and a ring fills. When it completes, Rejoin selects it, no tap needed. The tiles are large and calm, tuned for Switch Control and eye tracking.

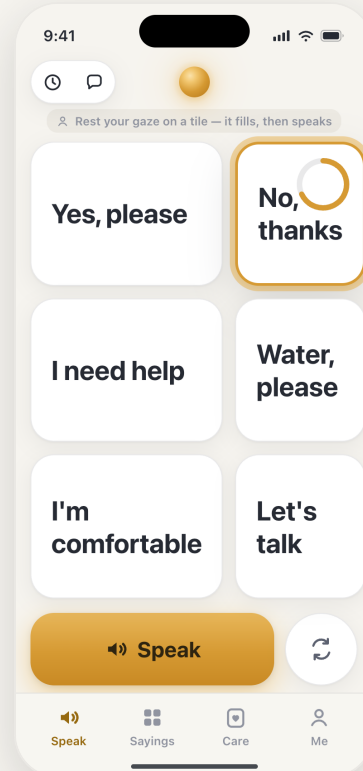

Dwell tiles.


3. Head gestures

A nod can mean yes and a shake no; a head tilt can speak what is typed. A test-and-calibration screen (Me, then Assisted input, then Head gestures) shows a live face indicator and a Gentle, Standard, or Firm sensitivity so it fires when meant and stays quiet otherwise.

4. Turning it on

Open the Me tab and go to Accessibility and display. Enable dwell, head gestures, or, on Face ID devices, enhanced face tracking for blink and expression control. Start with one, tune it, then add more.

Nothing leaves the device. The camera runs only while a feature is on, and no images are ever stored or sent.

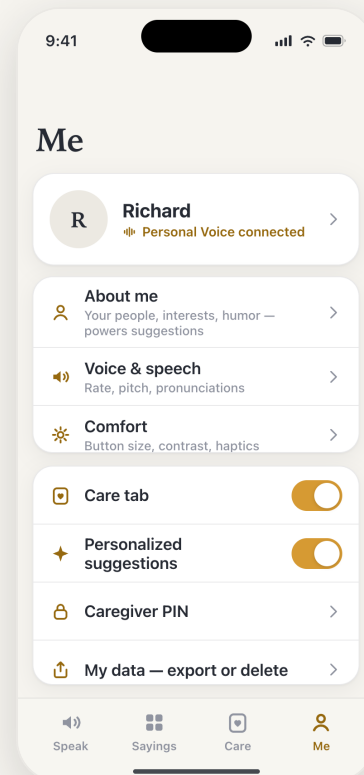

Me, then Accessibility.