



Guide 8

Speaking and live replies


The Speak tab: type, speak, and let Rejoin suggest replies.

The Speak tab is where most of Rejoin happens. You can type and speak, get help from AI, and, when you turn it on, see reply suggestions the moment someone asks a question.

For patients and caregivers.

1. Type and speak


Type in the box and tap the gold Speak button. Rejoin says it aloud in your chosen voice. Nothing is spoken until you tap.

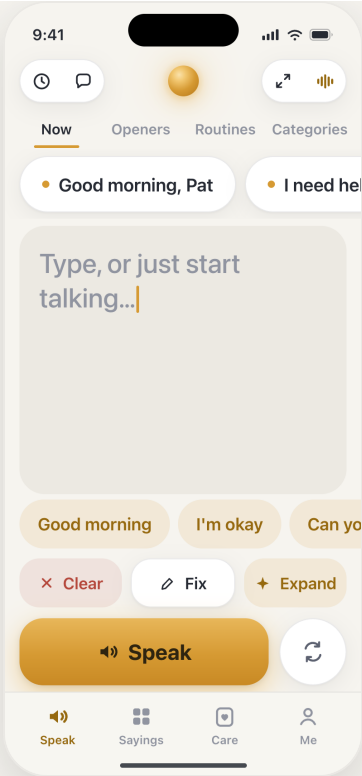

The Speak tab.

2. Live reply suggestions

Turn on listening and, as someone speaks, Rejoin lays a few replies in front of you, ready by the time they finish. Tap the one that fits. Listening runs on the device and audio is never stored.

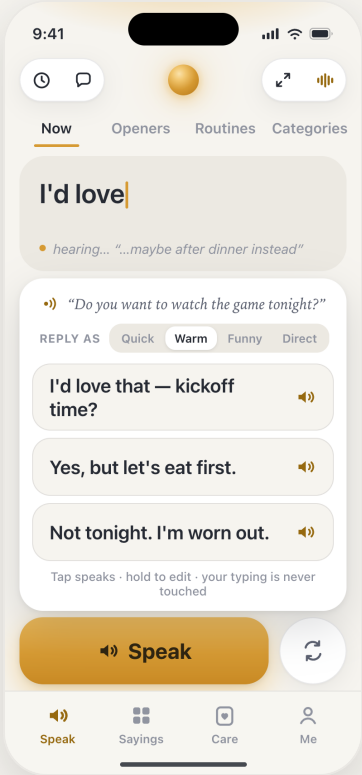


Replies as they talk.

3. Fix and Expand

Under the box, Fix tidies up a rushed message, and Expand turns a few words into a full sentence. Both are quick AI helpers, and you always review and tap Speak yourself.

4. Openers


Openers are questions you ask to start a conversation, so you are not only ever responding. Tap one to begin.

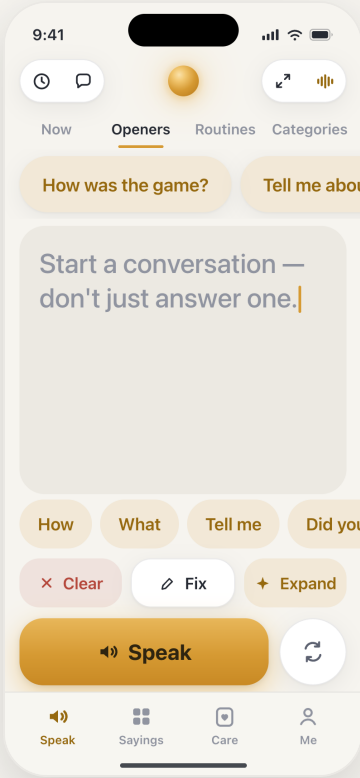

Openers.

5. Show it big

In a loud room, one tap fills the screen with your message and flips it to face the person across the table.

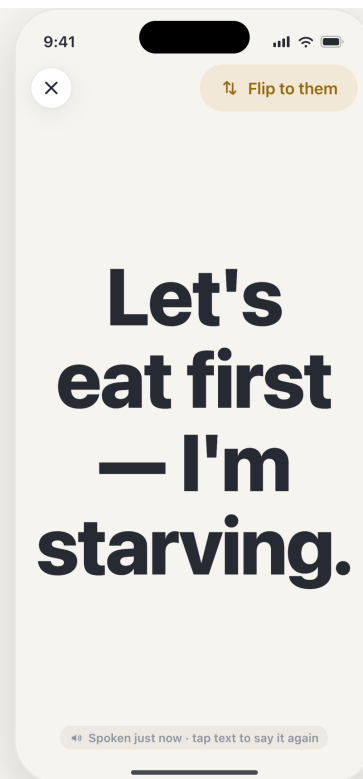

Display mode.